CARE

[REF]

**Textile, lightweight breast form
1014X, 1015X, 1018X, 1019X**


Gebrauchsanweisung. Instructions for use. Pokyny k použití. Brugsanvisning. Kasutusjuhend. Instrucciones de uso. Käyttöohje. Mode d'emploi. Upute za uporabu. Használati utasítás. Istruzioni per l'uso. Naudojimo instrukcija. Lietošanas pamācība. Arahan penggunaan. Gebruiksaanwijzing. Bruksanvisning. Sposób użycia. Manual de instruções. Instrucțiuni de folosire. Bruksanvisning. Návodilo za uporabo. Návod na použitie. Упътване за употреба. **使用说明.** Οδηγίες χρήσης. 取扱説明書. 사용 지침. Инструкция пользователя. ค่าแนะนำสำหรับการใช้งาน. ملخصات الاستخدام.

DE.....	3	FR.....	13	NL.....	23
EN	4	GR.....	14	NO.....	24
AR.....	5	HR.....	15	PL.....	25
BG	6	HU.....	16	PT.....	26
CN.....	7	IT.....	17	RO	27
CZ.....	8	JP.....	18	RU.....	28
DK	9	KR.....	19	SE.....	29
EE.....	10	LT.....	20	SI	30
ES.....	11	LV.....	21	SK.....	31
FI.....	12	MS.....	22	TH.....	32

REF

Textile, lightweight breast form
1014X, 1015X, 1018X, 1019X

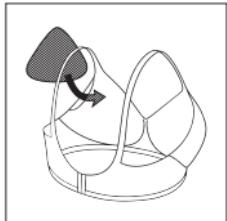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Instructions for use

Please inform the manufacturer – ANITA Dr. Helbig GmbH – and the responsible authority of any serious product-related incidents

Article

1014X, 1015X, 1018X, 1019X
Textile, lightweight breast forms

Indication

As initial/lightweight care after breast surgery (ablation/mastectomy, partial surgery)

Contraindications and risks

No contraindications or risk for the use of the products are known or can be expected. In some rare cases, allergic reactions may occur. Please contact your physician or ask medical specialist personnel if you are not sure whether or not you may be allergic, or if you have questions or issues with the use of these products.

Purpose/How it works

- Target group: Women after various types of breast surgery
- Initial/Lightweight care after breast surgery or during radiation therapy
- Skin-friendly, breathable material, reduces skin irritation on sensitive skin, e.g. during radiation therapy or stresses on the scar area

Materials

- Cotton, polyester, polyamid, polyurethane
- Latex-free
- See the garment label for material percentages
- ANITA products bear the OEKO-TEX® STANDARD 100, 98.0.1439 Hohenstein HTTI mark. It guarantees the absence of harmful or allergenic substances.

Care instructions

- Please see the garment label for care instructions.
- Remove the medical fleece before washing.
- Once the garment has dried, the primary care prostheses can be padded with the medical fleece once again.

User instructions

The right size for the textile primary care prosthesis depends on the cup size of the patient. Primary care prostheses can be worn in the pocket of a special textile. The volume can be individually adjusted by padding it with the medical fleece. Allow a trained professional to select the correct size for you.

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Technical data**Weight in grams
(without packaging or medical fleece)**

Sizes	1014X	1015X	1018X	1019X
60/65	22,0	18,7	14,8	14,5
70/75	35,5	26,8	20,3	19,4
80/85	45,2	33,4	26,4	25,0
90/95	51,5	39,0	31,0	30,0
100/105	62,5	49,5	38,9	37,5
110/115	91,5	57,2	47,0	45,4

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Designed and made by ANITA.

Thank you for choosing a high-quality product from ANITA.
The 100% family-owned business has been taking responsibility
for its products, employees and the environment since 1886.

ANITA has a certified quality management system that complies
with DIN EN ISO 9001 and DIN EN ISO 13485 to ensure the quality
of products throughout the entire value chain.

ANITA Dr. Helbig GmbH, Grafenstr. 23, D-83098 Brannenburg, www.anita.com

Corsa Nova AG/SA Anita Group, Wassergrabe 14, 6210 Sursee

ANITA UK Ltd., 23 Shirwell Crescent, Furzton, Milton Keynes, MK4 1GA